

Manufactured by: EZsurgical Ltd.,  
New Industrial Park  
POB 620, Yoqneam 20692 Israel  
Tel: +972-4-6665060  
www.ezsurgical.com  
info@ezsurgical.com  
marketing-bd@ezsurgical.com

EC REP

MedNet GmbH, BorkstraBe 10 48163 Munster  
Deutschland  
Tel: 49-251-32266-0  
Fax: 49-251-32266-22  
Email: nicole.boehnsich@medneteuropa.com


CE 0483

FDA 510(K) Number: K082291 patent-pending



## A-Lap™ The First 3D Laparoscopic Retractor


- 3-Dimensional "Hand-Like" soft tissue retractor
- Atraumatic, large volume tissue retraction
- Optimal visualization: Safe, Consistent Controlled
- Intuitive handle design, multiple lock-in angles

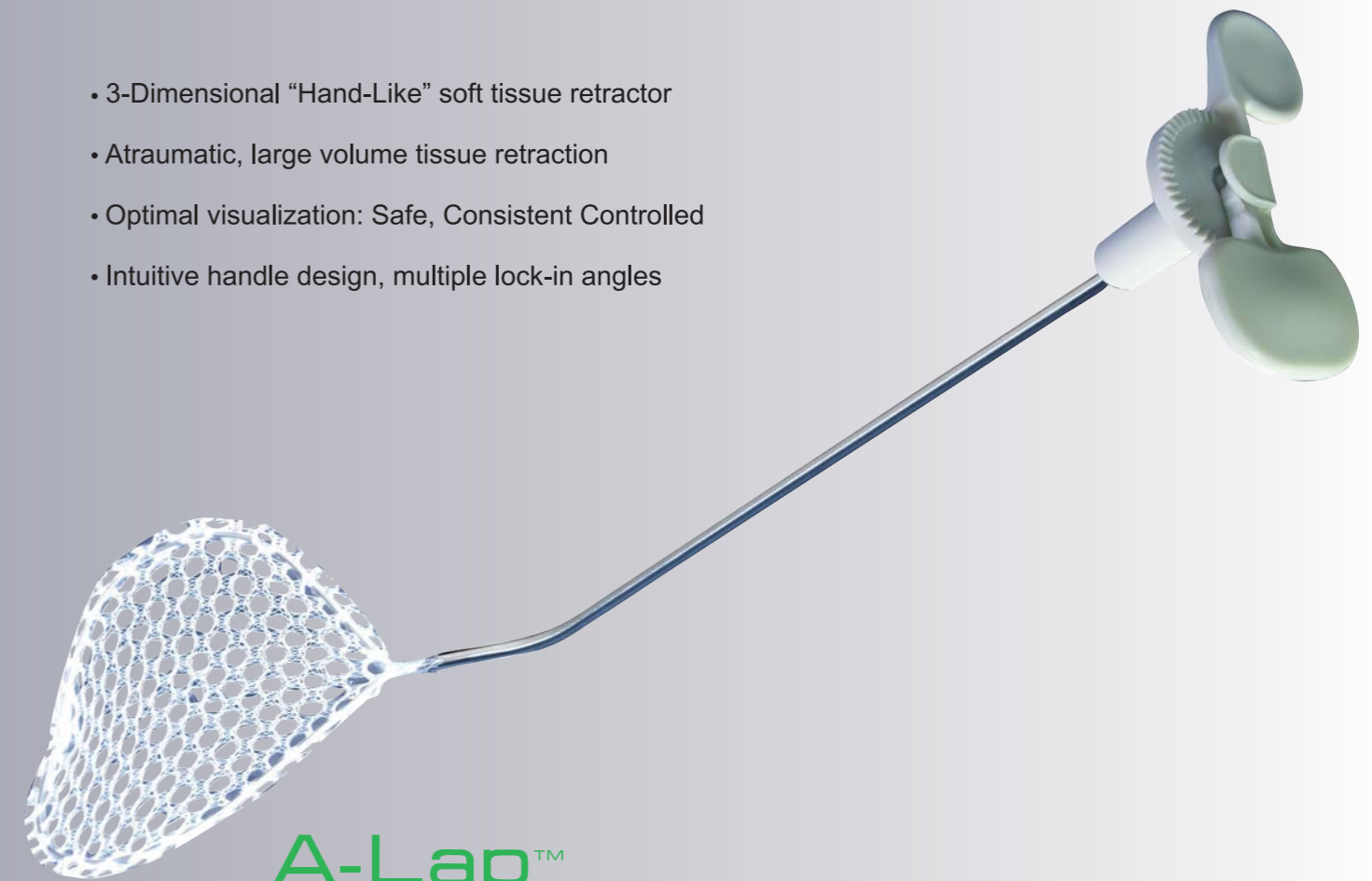

**A-Lap™**  
Simulating a Surgeon's Hand

# A-Lap™ Set


Advanced Simplicity

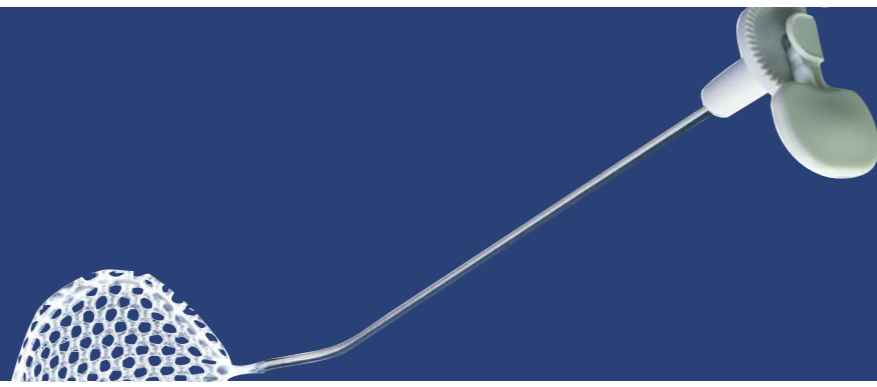

- The A-Lap™ set utilizes a unique, Hand-Like retractor designed to support the internal organs surrounding the operative site.
- The optimal soft tissue laparoscopic retractor for General Surgery, Gynecology, Urology.


## 1. Place

The EZaxess™ flexible port (included in set) is placed and used to facilitate introduction of curved laparoscopic instrument

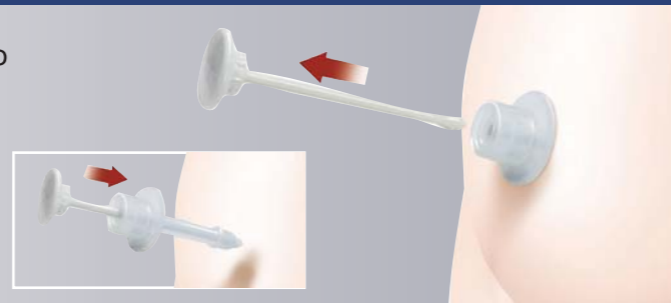


## 2. Insert

The A-Lap™ retractor is inserted through the port in a contracted form  
A-Lap™ may be used with newer flexible Single Port Access ports

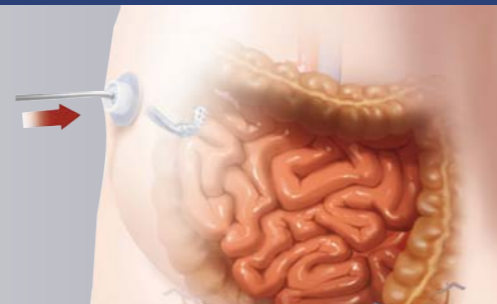


## 3. Open

Expansion is controlled by the surgeon utilizing an intuitive handle and locking mechanism

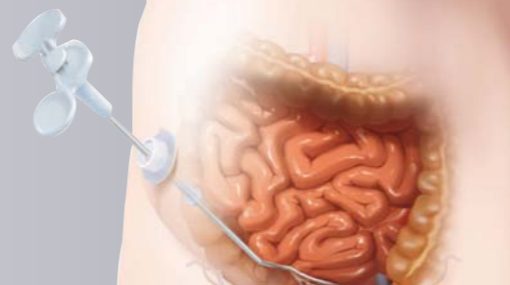


## 4. Retract

The A-Lap™ retracts tissue for optimal exposure of the operative site

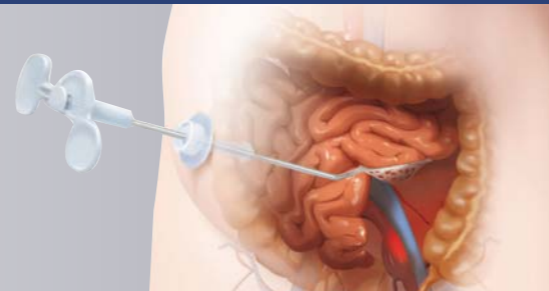


# A-Lap™

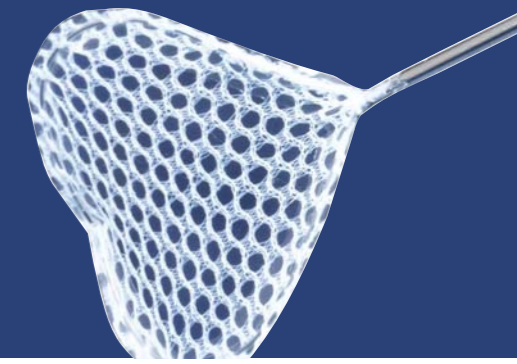

## A-Lap™ Curved



## The EZaxess™ Flexible Port

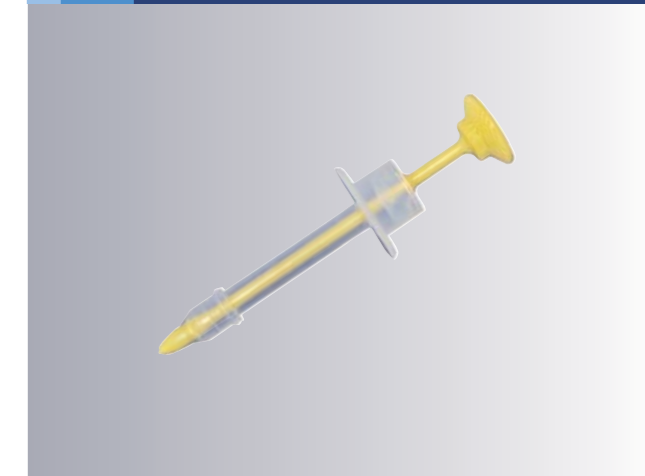

## EZaxess™ Flexible Port with A-Lap™



### Available Products

Description	Part Number	Barrier Diameter
A-Lap™ Curved Set Size 1	MAS0210-01	74
A-Lap™ Curved Set Size 2	MAS0220-01	92
EZaxess Flexible Port	MAS0102-00	-

- Each A-Lap™ Set is individually packaged, sterile, latex free and includes EZaxess (Flexible port)
- The master pack contains three individual A-Lap™ packages